



Delve Detect is a metagenomic sequencing (mNGS) test that detects RNA and DNA from bacteria, virus, fungi, and parasites in a single sample of CSF



References a database of **>68,000 pathogen species**

22%

22% greater diagnostic yield compared to all other infectious disease tests¹



Only **1 mL of CSF** sent in Delve Detect kit, **shipped at room temperature**



Results available 48 hours after sample received



Includes **expert medical consultation** via Clinical Microbial Sequencing Boards

97% PPV
92% NPV

For diagnosis of CNS infections based on 7 yrs of data on >4,800 patients¹

¹ Diagnostic performance over 7 years and >4800 CSF samples

Quick start guide

Delve's intuitive workflow enables labs to go from sample-to-answer simply and quickly.




Assemble Materials

- **Remove** all contents from the kit.
- **Read** the instructions for use.
- Do not discard Stabilization buffer.



Complete and Fax Requisition Form

- **Review** the provided Test Requisition Form and complete all required fields.
- **Affix** your institution's specimen label and the Delve Barcode Label to the top of the form (within the dotted lines).
- **Fax** Requisition Form to 1-833-556-5216.

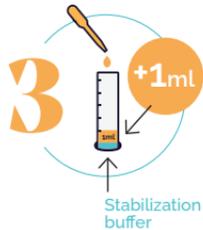

Transfer Sample

- Under clean conditions, **add 1 mL of CSF** into the Delve Tube.
- **Close** tube tightly, place into Secondary Safety Bag. Seal the bag.



Safety Bag

- **Place** Requisition Form and filled Delve Tube (in safety bag) into the box.
- **Peel** the adhesive strip and press to **seal** the box.



Ship Box

- The kit box is **prelabeled** with overnight shipping.
- **Place** box at designated pickup location.

Results will be available 48 hours from sample receipt.